PRESSALIT®

R1171 (R1170)  
R1191 (R1190)

|                                                                      |           |
|----------------------------------------------------------------------|-----------|
| <b>TMA arm support</b> – Operation and maintenance manual            | <b>en</b> |
| <b>TMA toiletstøtte</b> – Brugs- og vedligeholdelsesvejledning       | <b>da</b> |
| <b>TMA Stützklappgriff</b> – Gebrauchs- und Pflegeanleitung          | <b>de</b> |
| <b>TMA barre de relèvement</b> – Manuel d'utilisation et d'entretien | <b>fr</b> |
| <b>TMA toiletsteun</b> – Gebruiks- en onderhoudshandleiding          | <b>nl</b> |
| <b>TMA toalettarmstöd</b> – Bruks- och underhållsanvisning           | <b>sv</b> |
| <b>TMA toalettstøtte</b> – Bruks- og vedlikeholdsveiledning          | <b>no</b> |
| <b>TMA apoyabrazos</b> - Instrucciones de uso y mantenimiento        | <b>es</b> |
| <b>TMA podpora</b> – Instrukcja obsługi i konserwacji                | <b>pl</b> |
| <b>TMA 臂撑</b> - 操作和维护手册                                              | <b>中文</b> |

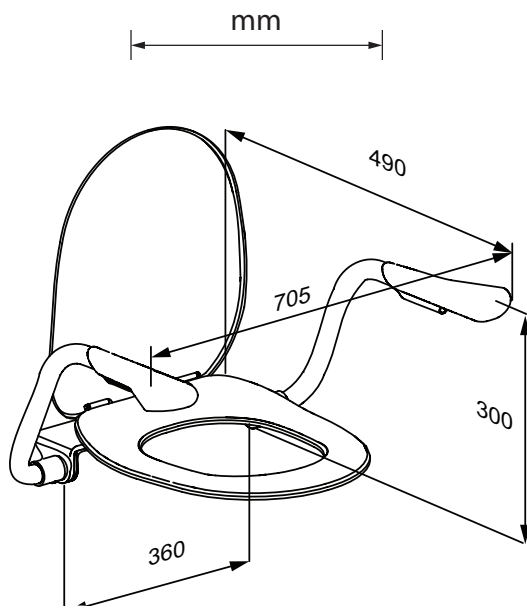
|              |    |
|--------------|----|
| English      | 6  |
| Dansk        | 15 |
| Deutsch      | 24 |
| Français     | 33 |
| Nederlands   | 42 |
| Svenska      | 51 |
| Norsk        | 60 |
| Español      | 69 |
| Język polski | 78 |
| 中文           | 87 |

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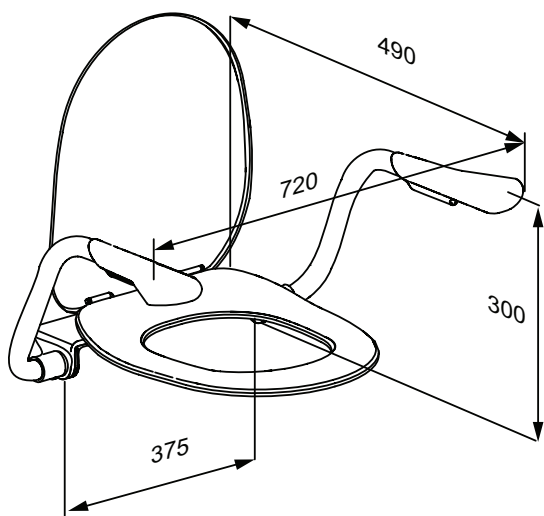
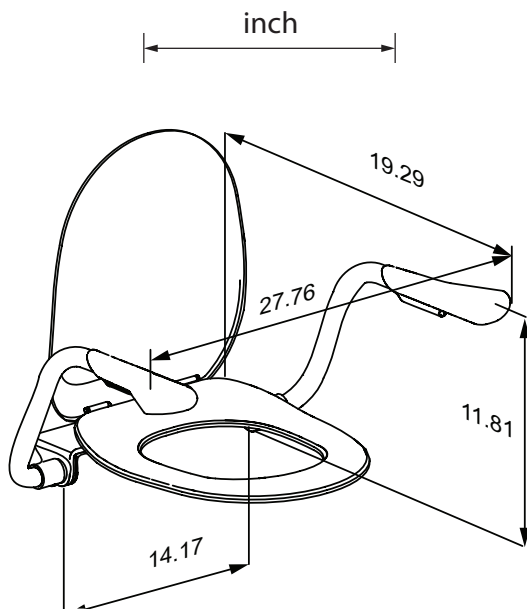
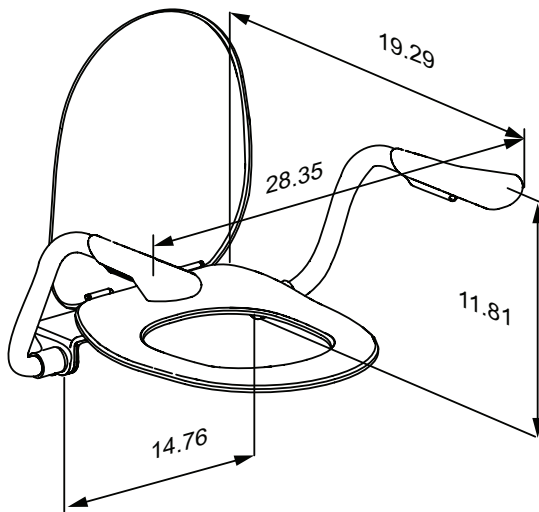
R1191 (R1190)

# Content

|                                   |    |
|-----------------------------------|----|
| Mounting instruction .....        | 6  |
| Symbols used in this manual ..... | 7  |
| General safety .....              | 8  |
| Product label .....               | 9  |
| Intended use .....                | 10 |
| Intended user profile .....       | 10 |
| Operating environment .....       | 10 |
| Operating instructions .....      | 11 |
| Maintenance .....                 | 12 |
| Maintenance inspection .....      | 12 |
| Cleaning .....                    | 13 |
| Technical data .....              | 13 |
| Disposal and recycling .....      | 14 |

en

## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [pressalit.com](http://pressalit.com).

Original instructions

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

Hand injury warning

Product weight

Max. user weight

Article number/  
versions

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

## **CAUTION**

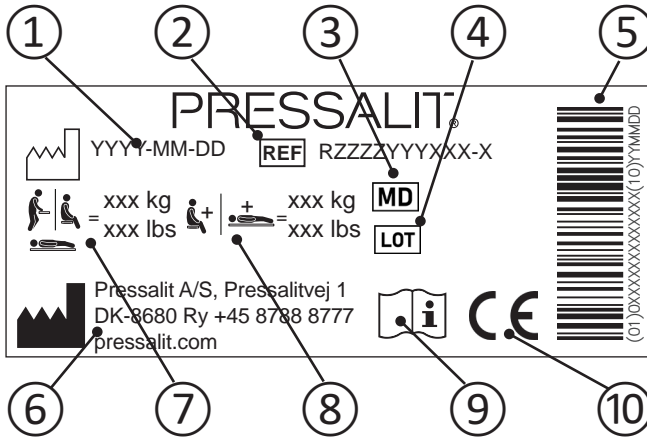
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at [pressalit.com](http://pressalit.com).

## Intended use

The arm support is used as support for a person when he or she sits down or gets up from a toilet.

## Intended user profile

The product can be used by all - adults, children, young and old.

## Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years.

*Life cycle test performed according to ISO 17966:2016.*

The surface of the product may become hot and/or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Fire hazard. Keep open flame away from the grey handles.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

# Operating instructions

The two arm supports can be folded up and down individually. Fold them both up when not in use.

en

**Take care to avoid injury to hands when in the vicinity of closing moveable parts.**

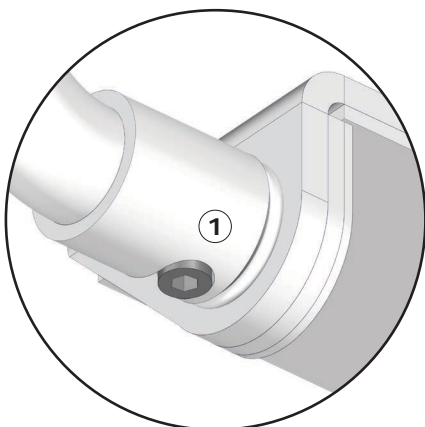
# Maintenance

It is recommended that maintenance inspection is done once a year.

## Maintenance inspection

Check the screws ① for wear.  
Replace them if necessary.

Do not use hinges other than  
the original ones.

**CAUTION**

- Do not use the product if it is broken or damaged.
- Do not repair items not described under maintenance.

# Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on [pressalit.com](http://pressalit.com).

## CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

## IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

### Technical data

|                 |                     |
|-----------------|---------------------|
| Max user weight | 140 kg /312 lbs     |
| Materials       | Steel, Polyurethane |

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

|              |                                                                                                                                                                                                                                             |
|--------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Steel        | Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.                                                                                                                               |
| Polyurethane | Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents. |

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